



# X-Ray Software Onboarding

To switch to VoyanceX acquisition software and VPAX CLOUD, please see the implementation process below:

## Step 1: Verification

Provide a photo of your DR panel. This is needed to identify the manufacturer, model, and manufacture date to cross-reference compatibility with VoyanceX.

— **Option A:** DR Panel Compatible with VoyanceX  
Note: Allow for a week until going live for in-clinic use.

1. Download and configure VoyanceX acquisition software and set up the VPAX CLOUD upload.
2. Migrate and upload the existing image archive to VPAX CLOUD.
3. Conduct training: the Voyance team will provide one hour of training on VoyanceX acquisition and workflow and will handle the VPAX CLOUD training.
4. Go live with in-clinic use.

— **Option B:** DR Panel Not Supported by VoyanceX

1. Access the 3rd-party acquisition software to configure a new DICOM send destination for the VPACSmini Gateway.
2. Migrate and upload the existing image archive to VPAX CLOUD through the Gateway.
3. The Voyance team will train the staff on VPAX CLOUD workflow.
4. Go live immediately after training.

